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GB Installation instructions

Attach the 2 x L-shaped brackets to the raised toilet seat; turn the model upside down on a flat surface and insert the L-shaped brackets into the slots. Push gently on the bracket and turn the knob clockwise until the thread engages. Position the brackets leaving a greater width than that of the toilet bowl.

Lift up existing toilet seat and place the seat directly onto the toilet bowl positioning the brackets towards the rear and the lip against the front of the bowl. Turn the adjustment knobs clockwise until a resistance is felt and the seat does not move from side to side. Ensure seat is firmly fixed before using.

ATTENTION! The side-locks have been especially designed to allow for easy therefore turning be continued once a resistance is felt.

Cleaning Instructions
All parts can be cleaned with warm water and a neutral cleanser. Alternatively, the seat can be totally immersed in water and/or can be sanitized (without brackets) in an autoclave at 85°C for 1 minute.

Cautions
Do not exceed maximum user weight: 225kg /496lb Always ensure the toilet seat is securely fixed before using. Do not hold onto seat for support while rising/lowering.

Bidet
Portable bidet for use with raised toilet seats. Fits snugly allowing easy transfer. Soap dish located at the front to assist personal cleaning.

User Instructions
Fill bidet with warm water and place inside installed raised toilet seat. Use as normal bidet. Tip used water into toilet bowl.

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